



LIMITED LIFETIME WARRANTY

Revolution Mobility is focused on providing quality products. This product is designed and built to precise standards and is carefully inspected prior to shipment. This product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This warranty does not cover product failures due to owner misuse or negligence or normal wear and tear. This warranty does not extend to non-durable components such as rubber accessories which are subject to normal wear and tear and does require periodic replacement. In the event of a defect, Revolution Mobility will at our option replace or repair the product.

If you have any questions please contact an authorized Revolution Mobility Dealer.

TRAPEZE



3072 Inland Empire Blvd. • Ontario, CA 91764
866.980.2259 • 909.980.2269 fax
www.RevolutionMobility.com



INSTRUCTIONS AND WARRANTY



TRAPEZE

PLEASE READ THIS MANUAL CAREFULLY TO ENSURE SAFE USAGE AND MAINTENANCE

How to operate and maintain

Assembling the Trapeze

1. Remove shipping end caps from A-frame.
2. Insert the base assemblies into the A-frame.
3. Depress the snap button on one (1) base assembly and slide it into the A-frame until the snap button protrudes through the hole in the A-frame.

NOTE: The base assembly will lock when an audible click is heard.

4. Repeat STEP 3 for the other base assembly.
5. Slide the mast and boom tube assembly onto the A-frame, with the boom tube on the same side as the base assembly legs.
6. Remove the boom tube end cover from the boom tube.
7. Slide the trapeze clamp over the end of the boom tube.
8. Depress the snap button on the boom tube and slide the trapeze clamp toward the mast until it locks.

NOTE: The trapeze clamp will lock when an audible click is heard.

9. Replace the boom tube end cover.
10. Adjust the trapeze. Refer to Adjusting the Trapeze.

Adjusting the Trapeze Height

1. Position the desired chain link onto the free S hook on the trapeze clamp (Detail A).

Position

1. Loosen trapeze knob by turning counterclockwise (Detail A).
2. Slide the clamp to desired position on the boom tube.
3. Secure the trapeze clamp to the boom tube by turning the knob clockwise.

Rotation

NOTE: The mast and boom tube can be rotated to three positions.

1. To adjust the rotation, pull out the quick release pin (Detail B).
2. Move the trapeze to the desired position. There are three (3) holes to choose from as shown in Detail C.
3. Let go of the quick release pin once the trapeze is in the desired position. You should hear an audible click as the quick release pin enters one of the three (3) holes and locks into place.

Trapeze Placement

1. Position the trapeze at the head of the bed so the boom tube hangs over the bed.

FIGURE 1 Assembling the Trapeze

