
**Tub Safety Bar**  
**Barre d'appui pour la baignoire**  
**Barra de sujeción para tina**

CP2968-XXX-01

**REQUIRED TOOLS:**  
 Wrench (provided)  
 Standard size slotted head screwdriver

**INSTRUCTIONS:**  
 Before using this product, it is recommended to consult a health care professional for your specific needs. Both inside and outside of the tub mounting wall area must be clean and dry prior to installation.

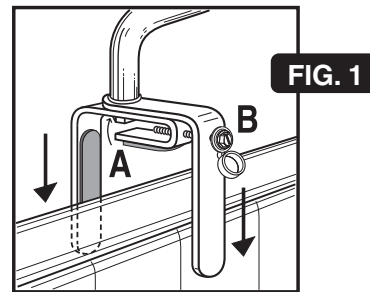
1. Open (unsnap) plastic cover on bolt **B**. Using the wrench provided, loosen bolts **A** and **B**, then slip clamps downward on tub walls. Be sure that all three (3) gripping pads inside each clamp meet both sides and top of the tub wall. Bolt **B** on clamps must face to outside of tub. (See FIG. 1)
2. Using the wrench provided, tighten bolt **A** (only enough to close gaps between the safety bar and clamps). (See FIG. 2)
3. Tighten bolt **B** firmly and securely (do not overtighten) so that all three (3) gripping pads inside each clamp make continuous contact with both sides and top of the tub wall. Securely retighten both bolts **A** and **B**. Close plastic cover on bolt **B**. (See FIG. 3)

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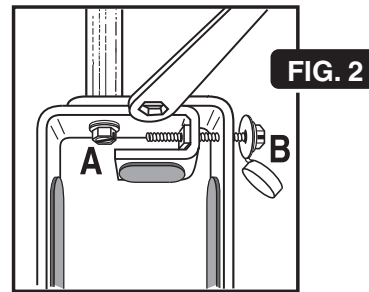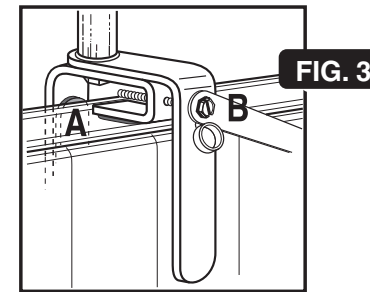
**WARNING / ATTENTION / ADVERTENCIA**

- This Tub Safety Bar should not be used by persons weighing more than **250 pounds**.
- DO NOT install this product over a tub wall with a shower door track.
- This Tub Safety Bar will not clamp securely to all bathtub designs.
- This Tub Safety Bar will securely attach only to bathtubs with parallel inner and outer walls. It will NOT securely attach to bathtub walls that are angled. (See FIG. 4)

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FIG. 4

- This Tub Safety Bar will NOT securely attach to tubs whose inner and outer walls are rippled or wavy (with design relief variations). (See FIG. 5)

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FIG. 5

- Make sure the gripping pads on the clamps make continuous and firm contact with both the inner and outer walls of the bathtub. (See FIG. 6)

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FIG. 6

**WARNING / ATTENTION / ADVERTENCIA**

Tub Safety Bar may accidentally pull off the tub when user is exiting the tub if:  
 • Tub Safety Bar is not securely attached  
 OR  
 • Tub Safety Bar is used by persons who weigh more than **250 pounds**.  
 This may result in an accidental fall that could result in serious personal injury! Liberty Hardware Manufacturing Corporation shall have no liability for any damage or injury caused by improper installation or use of this product.

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**IMPORTANT INFORMATION:**  
**Specifications:** Tub Safety Bar will mount on tub wall widths from 3-3/4 in. to 5-3/4 in.

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**CLEANING INSTRUCTIONS:**  
 To clean, wipe with soft damp cloth. Do not use detergents or cleansers that may be harmful to the finish.

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